# PACEY CUFF™ TURBO

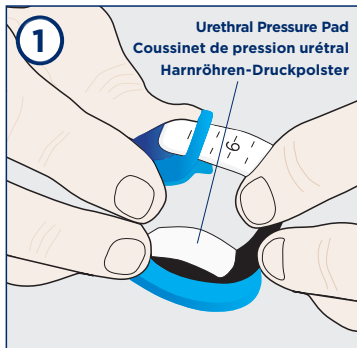
URETHRAL CONTROL DEVICE  
APPAREIL CONTRE L'INCONTINENCE URINAIRE  
GERÄT ZUR HARNRÖHRENKONTROLLE

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Thread the Velcro® strap through the guide on the top hood.

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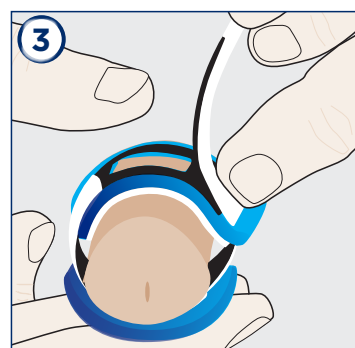
Slide over penis to rest at the base. Press top hood and bottom urethral compressor together with your dominant hand. Ensure the top hood and bottom compressor are centered.

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
While maintaining compression, pull up on the Velcro® strap with opposite hand to tighten. Attach strap to the top hood.

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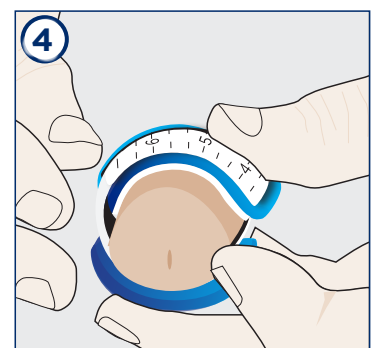Secure the remaining Velcro®. Ensure top hood and bottom compressor are centered on the penis. Inspect and treat any skin pinching that may have occurred between upper and lower parts.

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Symbol	Definition
	Manufacturer
	Authorized representative in the European Community
	Consult instructions for use
	Batch code
	CE Mark
	Product code

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Pacey MedTech Limited | Disruptive Medical Innovation  
Medical Device Development Center | 101 - 1099 West 8th Avenue  
Vancouver, BC Canada V6H 1C3 | Phone: +1-877-377-2239  
info@paceymedtech.com | paceycuff.com

Pacey MedTech Limited | Disruptive Medical Innovation  
Medical Device Development Center | 101 - 1099 West 8th Avenue  
Vancouver, BC Canada V6H 1C3 | Téléphone : +1-877-377-2239  
info@paceymedtech.com | paceycuff.com

Pacey MedTech Limited | Disruptive Medical Innovation  
Medical Device Development Center | 101 - 1099 West 8th Avenue  
Vancouver, BC Canada V6H 1C3 | Telefon: +1-877-377-2239  
info@paceymedtech.com | paceycuff.com

European Representative  
Medical Device Safety Service GmbH  
Schiffgraben 41  
30175 Hannover  
Germany

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Patent pending. Pacey Cuff (PC),  
Urethral Control Device (UCD),  
Urethral Pressure Pad (UPP)  
are trademarks of  
Pacey MedTech Ltd. ©2018  
Doc: PMT IFU EN-FR-DE Rev. 9.2

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## INSTRUCTIONS FOR USE

The Pacey Cuff™ is indicated for provision of male urinary incontinence. The device may be used during waking hours as required. Light pad use is recommended during the day and at night.

Voiding should be every 1-2 hours and light fluid intake is recommended.

The device may be worn during exercise and swimming.

### 1. The Pacey Cuff™ has 3 parts:

- The fenestrated hood (top) which protects circulation.
- The compression piece (bottom), which applies pressure to the urethra.
- The Velcro® strap.

### 2. Sizing:

Patients measure their penis circumference to determine sizing (See Sizing Chart).

### 3. Assembly:

The Velcro® strap is fed into the slot as shown (Fig 1).

### 4. How to apply the Pacey Cuff™:

- Ensure the top hood is centered on the top of the penis. (Fig 2)
- Ensure the bottom pad is centred on the bottom of the penis (in line with urethra). (Fig 2)
- Squeeze the top and bottom pieces together, applying firm pressure to the bottom, compressing the urethra. (Fig 2)
- The Velcro strap should be applied to the hood to maintain the pressure. (Fig 3)

**NOTE:** Compression and tightening of the Velcro® strap may need to be attempted a few times to determine the correct compression needed to occlude the urethra.

- Affix the top strap at desired tension. Number should be lined up at the centre. (Fig 4)
- This number is then used for day application. The pressure may be reduced at night as required.

**Night use:** Will require one (1) or more night voiding visits.

**Cleaning:** Wash the Pacey Cuff™ in warm water.

**Product life cycle:** Pacey Medtech™ recommends that the device be replaced as required (approximately every 4-6 months).

**Recommended:** The use of brief-style undergarments (form-fitting) rather than looser boxer shorts.

### Warnings:

- Remove the Pacey Cuff™ if any pain, swelling, bleeding or discomfort occurs.
- If patient has special needs, use the device in conjunction with professional supervision.
- To avoid cross contamination, the Pacey Cuff™ should only be used by one person.
- Do not use the device if you are on any medication that impairs your ability.
- If skin becomes irritated, discontinue use until the irritation resolves.
- If you have advanced peripheral or cardiac vascular disease or diabetes mellitus, please consult your physician or medical professional prior to use.
- If you have a urinary tract infection (UTI) please consult your physician or medical professional before use.

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