


C₃ With its unique anatomical design, C₃ advances external continence control over that provided by other devices

C₃ provides men with an easy-to-use, comfortable way to gain control and confidence over incontinence

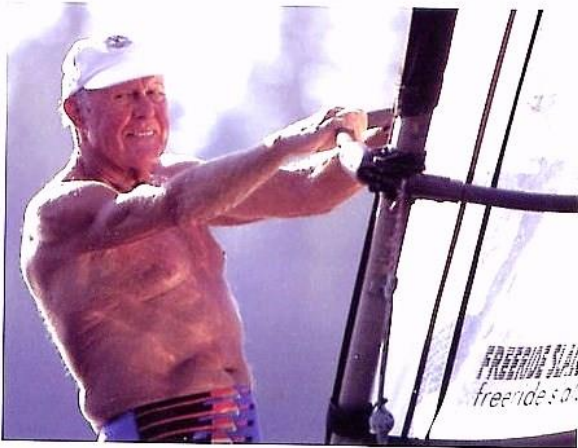

Comfort with C₃

The contoured, cradle-like design and elastic strap of the C₃ minimizes excessive pressure by distributing the radial force evenly around the penis. Carefully chosen materials also enhance your comfort.... Soft foam wrapped in supple plastic.

Available in two sizes, the C₃ offers a customized fit for even greater comfort.

Control with C₃

When the C₃ is placed on the penis, a urethral occlusion pad presses on the urethra. While this localized pressure prevents the involuntary loss of urine, the flow of blood is unimpeded to assure your safety.

Confidence with C₃

So compact, the C₃ is easy to conceal. The elastic strap holds the device in position, allowing you to go about your daily routine without concern about the C₃ shifting.

Proper hygiene never has to be a concern either. The disposable C₃ is reasonable priced, so you can replace the device at regular intervals. The C₃ is a more economical choice than absorbent products.

Easy to use

The penis is placed through the opening in the C₃ (see the left picture).



The lower arm is positioned so that the urethral occlusion pad is against the urethra, and the device is secured by wrapping the elastic strap comfortably and snugly around the penis

(see the right picture).

The C₃ device comes in two sizes, and you may easily determine which is your size by using a simple tape measure.

For best results, conduct sizing when your body is at room temperature.

For example, don't conduct the sizing immediately after shower.

Sizing Instructions:

1. Please use a flexible tape measure.
2. Measure only a non-erect penis.
3. Wrap the measuring device around the penis, one to two cm from the body.
4. With the measuring device snug around the penis, read the number in cm. Do not pull tightly.
5. Note the measuring number when ordering:

Regular: from 6 to 10 cm Penile size

Large: from 10 to 14 cm Penile size

The C₃ Male Continence Device is sent to you in discreet packaging!

C₃

Male Continenence Device



Do you have still questions? Please call!

Distributed by:



Waffenschmiedweg 30a | 86911 Diessen
Germany

Tel.: +49 (0)8807-91138

Fax: +49 (0) 8807-940899

Internet: www.mtm-med.com

Manufacturer:



Redmond WA 98052 | USA

US Patent Number 5.184.629

C₃

Male Continenence Device



A Solution
That Offers
Comfort,
Control
and
Confidence



&



C₃

Male Continenence Device

a external, non surgical incontinence
management

Manage Male
Incontinence
with



Comfort,
Control
and
Confidence

